

1

# AMPUTATED NON-HEALING DIABETIC FOOT ULCER HEALED

**By** Dr. Yuriria Aguilar Carsolio

**Place:** Clínica Comedem, Mexico

**Study Initiated on:** 08/2022

**Age of Patient:** 80 yrs

**Sex:** Female

**Type of Wound:** Diabetic, left foot amputation plus infection

## Wound Condition before SilverStream:

Patient with history of **Diabetes**, lead to hospitalization and surgery leading to **amputation** followed by **non-healing wound with infection** that went unattended and developed uncontrollably.

Post Surgical healing delayed with **non-healing wound with infection**.

## Previous treatment used:

**Amputation** followed by **hospitalization** with **antibacterials** along with **other dressings**.

## BEFORE SILVERSTREAM - WEEK 0



# AMPUTATED NON-HEALING DIABETIC FOOT ULCER HEALED

## SilverStream Treatment :

SilverStream was applied 10 ml once a day for 6 weeks followed by once a day dressing.

Within a week, the wound was clean with no signs of infection. Increased blood circulation observed, with granulation tissue covering the wound. Within 6 weeks, the foot was completely covered with fresh granulation tissue and the foot was almost completely healed.

No. of 500 ml SilverStream Bottles Used: 1



## Doctor Speaks:



*Fast healing, excellent results. The patient did not have any adverse reactions or discomfort.*

Wound Characteristics	Week 0	Week 6
Wound Size	16cm <sup>2</sup>	5 cm <sup>2</sup>
Signs of Infection	Yes	No
Granulation Tissue	No, 0%	Yes
Non-viable Tissue	Yes	No